



LAB instructions

Prepare the received *Capitainer qDBS* sampling card as below.



Peel-off tabs on the backside of the card.



Tear the peel-off tabs.



Insert a sharp tool, e.g. tweezers under the paper disc.



Carefully remove the paper disc.



Put the disc in a preferred vial, e.g. microtiter plate for subsequent elution.



Capitainer qDBS sample ready for analysis.



Capitainer AB
Norr Mälarstrand 54
SE-112 20 Stockholm SWEDEN
info@capitainer.se

capitainer.se

COPYRIGHT © 2020 CAPITAINER AB. ALL RIGHTS RESERVED